
Electric Double Breast Pump

— User Manual —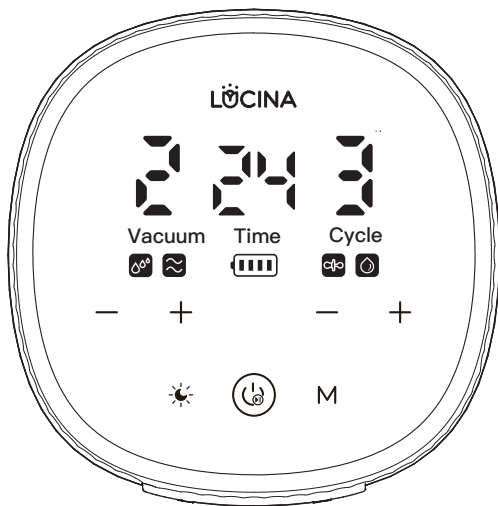

LED display · Four mode switch ·
Anti-back flow design · Lithium battery

COMPONENTS

Please check the following accessories before use, if there are missing or damaged pieces, please contact the supplier.

	Component Name	Materials	Operating Temperature	Sterilization Instruction
a	Upper cover	PP	-20°C/-4°F~120°C/248°F	Rinse in water/must dry out before using
b	Diaphragm	Silicone	-20°C/-4°F~200°C/392°F	
c	Pump body	PP	-20°C/-4°F~120°C/248°F	Boiling water/Microwave/Dishwasher/UV safe
d	Flange (24 mm)	PP	-20°C/-4°F~120°C/248°F	Rinse in water/must dry out before using
		PS	-40°C/-40°F~149°C/300°F	
e	Flange Insert (17/19/21mm)	Silicone	-20°C/-4°F~200°C/392°F	Rinse in water/Not suitable for prolong sterilizing
f	Duckbill valve	Silicone	-20°C/-4°F~200°C/392°F	
g	Bottle cover	PP	-20°C/-4°F~120°C/248°F	Boiling water/Microwave/Dishwasher/UV safe
h	Nipple	Silicone	-20°C/-4°F~200°C/392°F	Rinse in water/Not suitable for prolong sterilizing
i	Screw cover	PP	-20°C/-4°F~120°C/248°F	Boiling water/Microwave/Dishwasher/UV safe
j	Bottle	PP	-20°C/-4°F~120°C/248°F	
k	Motor	/	/	Wipe with soft duster
l	Air tube connector	/	/	
m	Air tube	Silicone	-20°C/-4°F~200°C/392°F	
n	Charging Cable	/	/	


HOW TO ASSEMBLE


Before use ensure your hands and the breast pump are cleaned.


1. Push the silicone duckbill valve onto the pump body.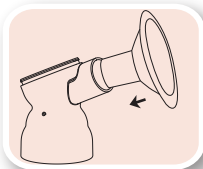

2. Attach the flange onto the pump body.

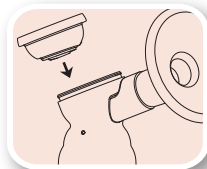
3. Put the diaphragm into the pump body.

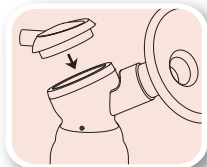
4. Screw the diaphragm cover onto the pump body.


5. Screw bottle into the pump body.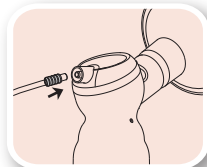

6. Connect both ends of the air tube with tube connector. Insert one end of the connector into the diaphragm cover.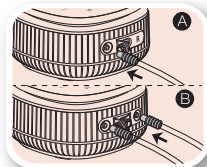

7. Single/Double sides switch: (A) insert one tube into one side and block the other side to use single pump; (B) insert tubes into both side to use double pump. (Note: Make sure air tube connector is correctly and fully inserted into motor)

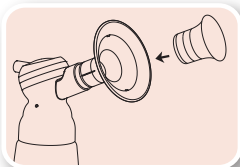8. 17/19/21 Flange insert is only suitable for use with 24mm Flange. Use a ruler or nipple measure card to find the right flange size.

9. After finishing pumping please store your breast milk in the refrigerator, then keep the electric parts dry.


Do not fill the bottle with too much milk, cannot pass the maximum capacity of the bottle, if too much milk in the bottle, milk will flow from the bilateral porosity of the pump body, please turn off the machine and stop sucking, replace the bottle to continue sucking. If you find it pass current bottle, please replace larger bottle.

For convenience, we especially equipped 2 extra wearable cups in the package. And the following part illustrates the components and assembling procedures of the equipped wearable cups.

COMPONENTS OF WEARABLE CUPS

Please check the following accessories before use, if there are missing or damaged pieces, please contact the supplier.

	Component Name	Materials	Operating Temperature	Sterilization Instruction
(a)	Flange Insert (21/24mm)	Silicone	-20°C/-4°F~200°C/392°F	Rinse in water/Not suitable for prolonged sterilization
(b)	Breast shield (28mm)	PP	-20°C/-4°F~120°C/248°F	Boiling water/Microwave/Dishwasher/UV safe
		PS	-20°C/-4°F~149°C/300.2°F	
(c)	Duckbill valve	Silicone	-20°C/-4°F~200°C/392°F	Rinse in water/Not suitable for prolonged sterilization
(d)	Collection cup (150mL/5oz)	PP	-20°C/-4°F~120°C/248°F	Boiling water/Microwave/Dishwasher/UV safe
		PS	-20°C/-4°F~149°C/300.2°F	
(e)	Diaphragm	Silicone	-20°C/-4°F~200°C/392°F	Rinse in water/Not suitable for prolonged sterilization
(f)	Cover	PP	-20°C/-4°F~120°C/248°F	Rinse in water/must dry out before using

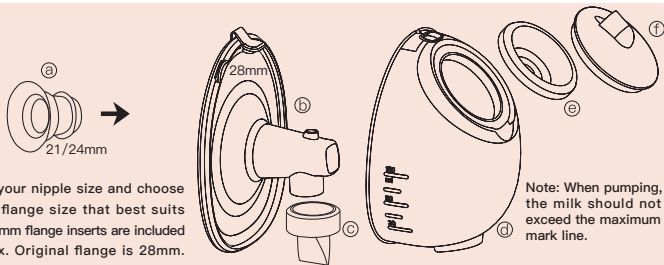
HOW TO ASSEMBLE THE WEARABLE CUPS


Before use ensure your hands and all breast pump parts are cleaned and dried.


Installation Video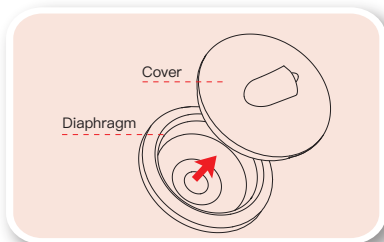


1. Put diaphragm into the upper cover, make sure there is no air leaking.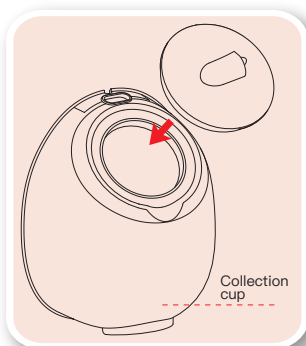

2. Press the installed upper cover onto the collection cup to ensure it fits in place.

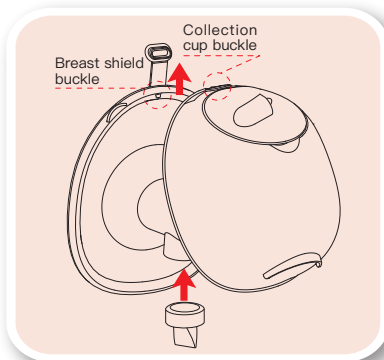
3. Push silicone duckbill valve onto the breast shield. Buckle up breast shield and collection cup on the top.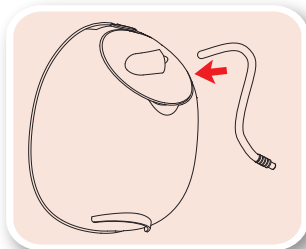

4. One end without the tube connector is inserted into the collection cup while the other end with connector is inserted into the motor.

DISPLAY AND BUTTONS

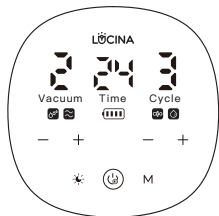Button use instruction

	On/Off Button	Long press to turn on/down. Short press to pause. The pump will automatic shut down after pause over 10 minutes.
	Mode switch button	Default multitronic mode "M" when device everytime turned on. Short press to switch modes.
	Night Light	When turn on the device, short press to turn the light on/off.
	Vacuum Adjust(left)	1 to 9 levels to adjust suction power
	Cycle Adjust(right)	1 to 5 levels to adjust suction frequency

Display Instruction

	Battery indicator	"" This symbol is flashing means the products should been charged as soon as possible. The device will auto shut down when the power is too low. The symbol will light up one by one while charging. Three symbols keep bright means the charge is finish and power is full.
 Vacuum	Level display	1 to 9 levels
 Time	Time display	The device will auto shut down when it keep work for 30 minutes.
 Cycle	Cycle display	1 to 5 levels
	Multitronic Mode	Variable frequency to get MORE milk.
	Massage Mode	Gentle STIMULATION to get milk flow.
	Suck Simulate Mode	Simulate NATURAL SUCKING of baby.
	Express Mode	Slow and deep sucking to EMPTY OUT .

FUNCTION FEATURES

The **vacuum level** and cycle speed can be adjusted independently by pressing “ — + ” button on the screen of the motor.

There are two groups of “ — + ” buttons on the screen.

The left side is to adjust the vacuum level and the right side is to adjust the cycle speed.

Vacuum Suction Setting—Adjust the vacuum level by pressing the “ — + ” button on the left side of the motor.

The vacuum suction level describes how aggressively the pump will “suck”; a higher number indicates more vacuum. Please be careful when setting the vacuum level on this breast pump. The pump has 1~9 vacuum suction settings. Some women may feel that higher vacuum levels promote faster milk expression. It is advised that you start with a low vacuum level and gradually increase it to a comfortable level because it is crucial to avoid setting the vacuum level so high that it causes discomfort, as this can affect your milk flow. You may discover that the level of vacuum that you are comfortable with varies greatly from day to day, throughout the day, and even during the same expression session.

Cycle Speed Setting—Adjust the cycle speed by pressing the “ — + ” button on the right side of the motor.

The cycle speed is measured in CPM (cycles per minute) on your breast pump and refers to how many times the pump “suckles” and releases per minute. This breast pump has been designed with 1~5 cycle speeds so that you can find the setting that suits your body best. For some women, altering the cycle speed can have a significant impact on their ability to express themselves. Since each woman is unique, try different speeds to see what suits you best.

VibesFlow Technology—Vibrate when pumping, Ensure efficient milk extraction while maintaining comfort. One highlights of this breast pump is that it adopts VibesFlow micro-vibration tech, which means the breast pump can vibrate at the same time to loosen the ducts when pumping. When the breast pump is working, it will stimulate the muscles and nerves around the breasts through micro-vibration massage to relieve the feeling of breast swelling and pain. At the same time, VibesFlow tech can promote blood circulation, so that mothers can feel more comfortable in the process of milking.

HOW TO CHOOSE MODE

Based on professional research and study, electric breast pump uses two-stage milk suction mode to stimulate the true rhythm that is closest to the baby's suction. There are four different modes and nine levels of suction. The intensity of suction increases with each level increasing.

 **Multitronic Mode:** Variable frequency to get **MORE** milk.

 **Massage Mode:** Gentle **STIMULATION** to get milk flow.

 **Suck Simulate Mode:** Simulate **NATURAL SUCKING** of baby.

 **Express Mode:** Slow and deep sucking to **EMPTY OUT**.

1 Min Massage Mode → Suck Simulate Mode “  ” → “  ”	Painless Pumping for Daily Use
1 Min Massage Mode → Express Mode “  ” → “  ”	Express Pumping for A Tight Schedule
1 Min Multitronic Mode → Express Mode “  ” → “  ”	Exclusive Combination for Oversupplyer
↓ ↓ ↓ There is a SPECIAL INGREDIENT for new moms who cannot produce much breastmilk at the beginning. Please relax yourself then try this combination below. ↓ ↓ ↓	
1 Min Massage Mode → 1 Min Multitronic Mode → Express Mode “  ” → “  ” → “  ”	Try it with a comfortable suction level

* How to use:

1. Ensure all parts of the breast pump were sterilized and installed correctly.
2. Please wash your hands and use a hot wet towel to massage your breast before use.
3. Press the flange firmly against the breast, leave no gaps and make sure your nipples are centered.
4. Keep it upright. Be careful not to distort the position of the product and the chest, otherwise the breast pump will have no suction.
5. Long press the power button to start. Choose the suction that makes you comfortable.

* Tips:

We suggest moms to try different modes and suction according to your actual breastfeeding situation.

Note:

This breast pump will automatic turn off after working 30 minutes.


TROUBLE SHOOTING

Too little or no suction

- A. Check whether diaphragm and diaphragm upper cover are sealed.
- B. Check whether the Flange at the bell mouth is installed in place.
- C. Check whether the Silicone duckbill valve is installed correctly and whether it is tilted.
- D. Check whether the connection parts between Air tube and host have been installed.
- E. Check whether the Air tube is twisted or pressed.
- F. Check whether the Air tube is loose at both ends with the main connector. If it is easy to fall off, cut off the loose Air tube and plug in the main connector to use it. After cleaning and disinfection of this diaphragm, it must be dried before use. Do not clean and disinfect Air tube (this tube is a snorkel and does not contact with breast milk).

When pumping, you should keep right posture, please do not lie on side in case of milk flows back.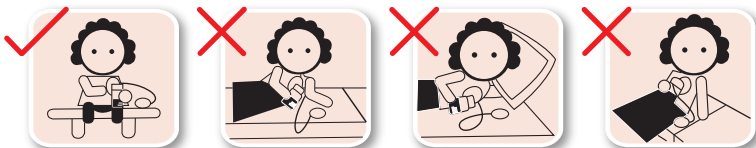

CAUTIONS

1. Please read the manual carefully before use.
2. Please fully charge the product before first use. Do not charge the battery pack for long time. Lithium batteries can only be replaced by or authorized service providers, and replacing the wrong model battery will have explosion dangers. Dispose the waste battery packs under local laws and regulations.
3. If the product is not used for a long time, the battery will run out of power. It needs to be plugged in for reactivation. After plugging in the power, wait for ten minutes for the screen to light up to indicate that it has been activated. Then continue charge the product till fully charged. If it is not activated for more than half an hour, please consult customer service.
4. The pump motor and other electric parts CAN NOT be washed in water or sterilized into steam sterilizer.
5. It is recommended not to disinfect the air tube and upper cover. If it is dirty, clean it and dry the inner and outer of the air tube. Make sure that there is no water droplets in the air tube before continuing to use breast pump. If there are water droplets in air tube, water droplets will be sucked into the motor, which will cause damage to the motor.
6. Please use the charging cable provided with this product. Do not plug the charging cable of any other manufacturer into the motor for use, so as not to cause damage to the motor.
7. Please use adapter, select the adapter with input voltage AC100~240V and output DC 5V and 3A.

Warning:

Please keep pump body, diaphragm, upper cover, air tube and other components dry to prevent the motor damaged from inhaling water!

POWER SUPPLY

Adapter: Insert the adapter into the power socket, long press power button to turn it on, choose mode and suction level. Short press power button to turn it off.

Lithium battery: This product has a lithium battery inside. When using it, long press power button to turn it on, choose mode and suction level. Short press power button to turn it off. When the battery symbol flashes, it means The power is low, please charge it in time.


CHARGING

When charging, pump should be turned off first. It will not be charging when turned on. Please plug adapter into the power socket. During the charging, the power symbol light will be flashing. When the battery symbol is stable bright, the battery is full.

Connect the charging cable to the USB port on the pump motor. breast pump includes an USB cable and can be charged by following ways:

1. Electric AC Adapter (input: AC100~240V/50Hz~60Hz; Output: DC5V/3A);
2. USB Power Bank (Output: DC5V/3A).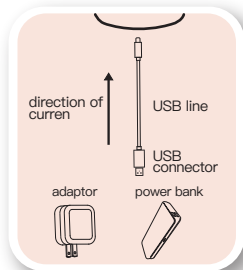

STORING BREAST MILK

Breast milk should be sealed and stored in refrigerator for long time storage.

Storage Ways	Storage Temperature	Longest Storage Time
Room	19°C~26°C	4~6 Hours
Refrigerator	<4°C/39.2°F	3~8 Days
Freezer	-15°C	14 Days
	-18°C	3~6 Months
	-20°C	3~6 Months

TO WARM MILK

1. In order to preserve the nutrition of the breast milk, you can warm the breast milk with 40°C/104°F warm water or use a bottle warmer.
2. Do not thaw frozen breast milk in the microwave oven or the boiling water (will damage the nutrition of breast milk).

WORKING CONDITION

Rated temperature range: 5°C/41°F to 40°C/104°F

Rated relative humidity: 20% ~ 80%

Rated atmospheric Pressure: 850Hpa ~ 1060Hpa

Long-term storage should be kept in a dry and cool place

TRANSPORT & STORAGE CONDITION

Temperature: -20°C/-4°F to 55°C/131°F

Humidity: 20% ~ 80%

Atmospheric pressure: 850Hpa ~ 1060Hpa

SAFETY CLASSIFICATION

properly dispose of the device in accordance with local, state and federal regulations Safety Classification.

—Type of protection against electric shock: Internally powered equipment.

—Degree of protection against electric shock: Type BF applied part.

—Safety degree of using in flammable anesthetic gas blending with air, oxygen or nitrous oxide: Non-AP/APG.

—Continuous or Non-continuous operation: Continuous.

—Classification of sterilization methods: No need for sterilization.

Symbol

Type BF
applied part

Stand by

Caution

PRODUCT SPECIFICATIONS

Name	Electric Double Breast Pump	Model	AM8132-1	Battery Specification	3.7 V 4400 mAh
Voltage	5V	Quantity	1pc		
Power	11W	Place of Origin	China	Warranty Time	1 year

WARRANTY CARD

Thank you for purchasing our product. Please keep your receipt as proof of purchase for any warranty claim.

Battery and motor can only be replaced by authorized party.

If you have any question,
please email us at lucinababy_amzus@hotmail.com.

Name		Telephone	
Product Name		Model	
Purchasing Date			
Address			

NOTICE

- 1.This product comes with 1 YEAR warranty. Please contact the MANUFACTURER for warranty claims.
- 2.This warranty does not cover defects caused by: misuse, abuse, improper storage/alteration of the product, use of improper components, failure to comply with the operating instructions using the accessories supplied with the product, or normal wear and tear.
- 3.The user of this product assumes all responsibility and risk for the use of this product.
- 4.WARNING: No modification of this equipment is allowed.

Jiangsu Xinbei Electrical Appliances Co., Ltd.

No. 115 Xinjin Road, Xinwu District,
Wuxi City, Jiangsu Province, China.

E-mail: lucinababy_amzus@hotmail.com

Please read the manual carefully before use.

Please store the product where children can not reach.

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Jiangsu Xinbei Electrodomésticos Co., Ltd.